





Technical Data

D-7172-SS


SWING AWAY GRAB BAR

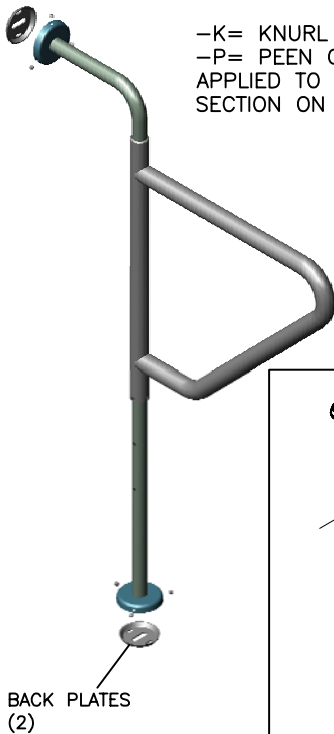

-K= KNURL GRIP
-P= PEEN GRIP
APPLIED TO TOP STRAIGHT
SECTION ON SUPPORT ARM.

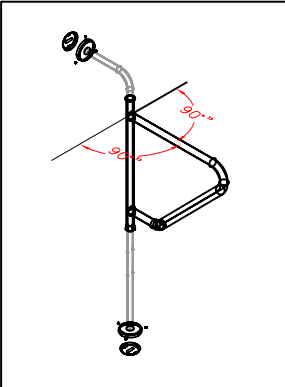

BACK PLATES
(2)

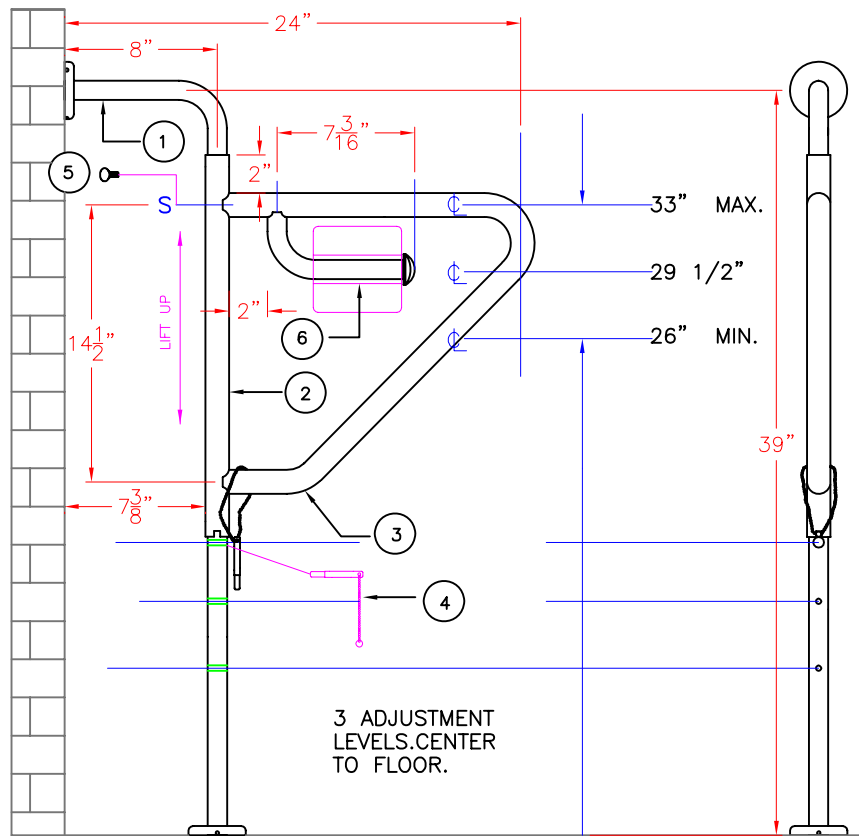

Construction:

MTG. TUBE: 1"Ø X .049W 304 S.S. TUBE.

SWIVEL TUBE: 1 1/4"Ø X .109W 304 S.S. TUBE, NOTCHED (4X) @90° FOR SWING ADJUSTMENT.


SUPPORT ARM: 1 1/4"Ø X .049W 304 S.S. TUBE WELDED TO SWIVEL TUBE.

FLANGES: 11 GA. CONCEALED MOUNT WELDED TO MTG. TUBE.

ADJUSTMENT PIN: MACHINED S.S, WITH CHAIN USE TO HOLD & LOCATE SWIVEL TUBE.

THUMB SCREW: USE TO LOCK SWING POSITION.

1	1" X .049W DIAMETER S.S.	1	MTG. TUBE
2	1 1/4" X .109W DIAMETER S.S.	1	SWIVEL TUBE
3	1 1/4" X .049W DIAMETER S.S.	1	SUPPORT ARM
4	ADJUSTMENT PIN W/CHAIN	1	
5	THUMB SCREW	1	
6	(OPTIONAL) TOILET PAPER HOLDER	1	

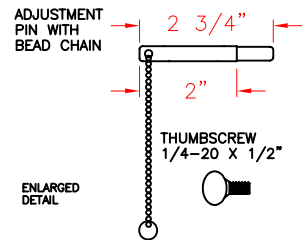

Operation:

ADJUSTMENT PIN IS USED TO HOLD SWIVEL TUBE AT DESIRED ORIENTATION.

THUMB SCREW LOCKS IN DESIRED ORIENTATION.

LOOSEN THUMB SCREW AND LIFT UP ON SUPPORT ARM. SWING TO NEW ORIENTATION AND SEAT ONTO ADJUSTMENT PIN AND NOTCH (ON SWIVEL TUBE).

TIGHTEN THUMB SCREW.

Installation:

USE BAR TO LOCATE AND MARK MOUNTING LOCATION.

FASTEN BACK PLATES (2) TO WALL AND FLOOR.

FASTEN BAR TO BACK PLATES USING PROVIDED SET SCREWS.

ADJUST SUPPORT ARM AS REQUIRED.

CAUTION: GRAB BAR IS ONLY AS STRONG AS THE ANCHORING DEVICES AND THE WALL & FLOOR ON WHICH IT IS MOUNTED TO.

Guide Specification:

CONCEALED MOUNT SWING-AWAY GRAB BAR.

304 STAINLESS STEEL.

SUPPORT ARM SWINGS 90° AND HELD IN POSITION WITH ADJUSTMENT PIN AND THUMB SCREW.

SATIN #4 FINISH WITH SAFETY GRIPS AVAILABLE.